

After inserting a new infusion set, the empty cannula needs to be filled with insulin. The amount of insulin required varies. Refer to instructions for use for your infusion set for specific instructions. Below are the cannula length and required insulin fill for Animas' most commonly used infusion sets.\*

|                        |                                                                                                                 |
|------------------------|-----------------------------------------------------------------------------------------------------------------|
| <b>Comfort™</b>        | 17 mm - fill empty cannula with 0.70 units insulin                                                              |
| <b>Comfort™ Short</b>  | 13 mm - fill empty cannula with 0.70 units insulin                                                              |
| <b>inset® 30</b>       | 13 mm - fill empty cannula with 0.70 units insulin                                                              |
| <b>inset®</b>          | 6 mm – fill empty cannula with 0.30 units insulin<br>9 mm – fill empty cannula with 0.50 units insulin          |
| <b>contact™ Detach</b> | No cannula fill required.<br>Prime infusion set until all medication dispenses from the tip prior to insertion. |
| <b>Other</b>           | Fill empty cannula with _____ units insulin                                                                     |

- Remember to only fill cannula when you insert a new infusion set.
- For a review of how to insert the infusion set, please go to: [www.infusion-set.com](http://www.infusion-set.com)
- For all infusion sets, consult the package insert and manufacturer's instructions

\* This information is provided for reference only. Always refer to the Instructions for Use provided with the infusion set to verify cannula fill requirements.