

# OneTouch® Ping® Factory Defaults

Setup Screen	Factory Defaults	Options
<b>Time</b>		
Hour	Mode: 12 Hr	12 or 24 Hr
<b>Sounds</b>		
A - Audio Bolus	H (High)	Vib (Vibrate), L (Low), M (Medium), H
N - Normal Bolus	H	OFF, Vib, L, M, H
R - Remote Bolus	Vib	OFF, Vib
T - Temp Basal	OFF	OFF, Vib, L, M, H
Alert	H	OFF, Vib, L, M, H
Reminder	H	Vib, L, M, H
Warning	H	Vib, L, M, H
Alarms	H	Vib, L, M, H
<b>Advanced</b>		
<b>Setup ADV 1 Audio Bolus</b>		
Enable	OFF	OFF, ON
Delivery Step	1.0 U	0.1 U, 0.5 U, 1.0 U, 5.0 U
<b>Setup ADV 2 Bolus</b>		
Adv. Bolus	OFF	OFF, ON
Reminders	OFF	OFF, ON
Delivery	NRML	SLOW, NRML
Basal Programs	1	1 or 4
<b>Setup ADV 3 Max LIMITS</b>		
Max Basal	5.00 U/Hr	0.00 U/Hr – 25.00 U/Hr
Max Bolus	16.00 U	0.00 U – 35.00 U
Max Daily	100 U	1 U – 600 U
2 hours	50 U	1 U – 100 U
<b>Setup ADV 4</b>		
Language	English	English, Spanish
Display timeout	30 secs	15, 30, 45, or 60 seconds
Contrast	8	1 – 10
Battery	Lith	Lithium or alkaline
<b>Setup ADV 5 AUTO OFF</b>		
Enable	OFF	OFF, ON
Suspend Duration	12 Hr	1 Hr – 24 Hr
<b>Setup ADV 6</b>		
Low Cartridge Warning	20 U	10, 20, 30, 40, 50 U
Occlusion Sensitivity	L	Low or High
<b>Setup Adv 7 (Ability to program 12 segments)</b>		
I:C ratio	12:00 AM, 1:15	1:2, 1:160
ISF	12:00 AM, 1:50	1:5, 1:600
BG Target	12:00 AM, 120 +/- 10	60 – 250 mg/dL, (+/-) 0 - 50 mg/dL
<b>Setup ADV 8 INSULIN on BOARD</b>		
IOB 2	OFF	OFF, ON
Duration	4.0 Hr	1.5 Hr – 6.5 Hr
<b>Setup ADV 9 Sick Days</b>		
BG over ____mg/dL	240	60 – 400 mg/dL
Check Ketones every ____Hrs	4 Hrs	1 Hr – 24 Hrs
Check BG every ____Hrs	2 Hrs	1 Hr – 10Hrs
<b>Setup ADV Meter</b>		
RF	OFF	OFF, ON
Search	----	ON
Channel	Auto	1 to 16